User manual

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Fidi
breast pump
laktator

EN
PL
DE

lionelo
care

Introduction

Dear customer!

Thank you for placing trust in us and choosing Lionelo.

Our products are manufactured with utmost care to provide your child with comfortable use and safety. Thanks to the use of high-quality materials and modern solution, we provide you with a product perfect for everyday use.

Should you have any remarks or questions concerning the product, please do not hesitate to contact us:

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IMPORTANT

Please read this manual before you start using the product to see all its functions and use them in accordance with intended use.

Keep this manual for further reference.

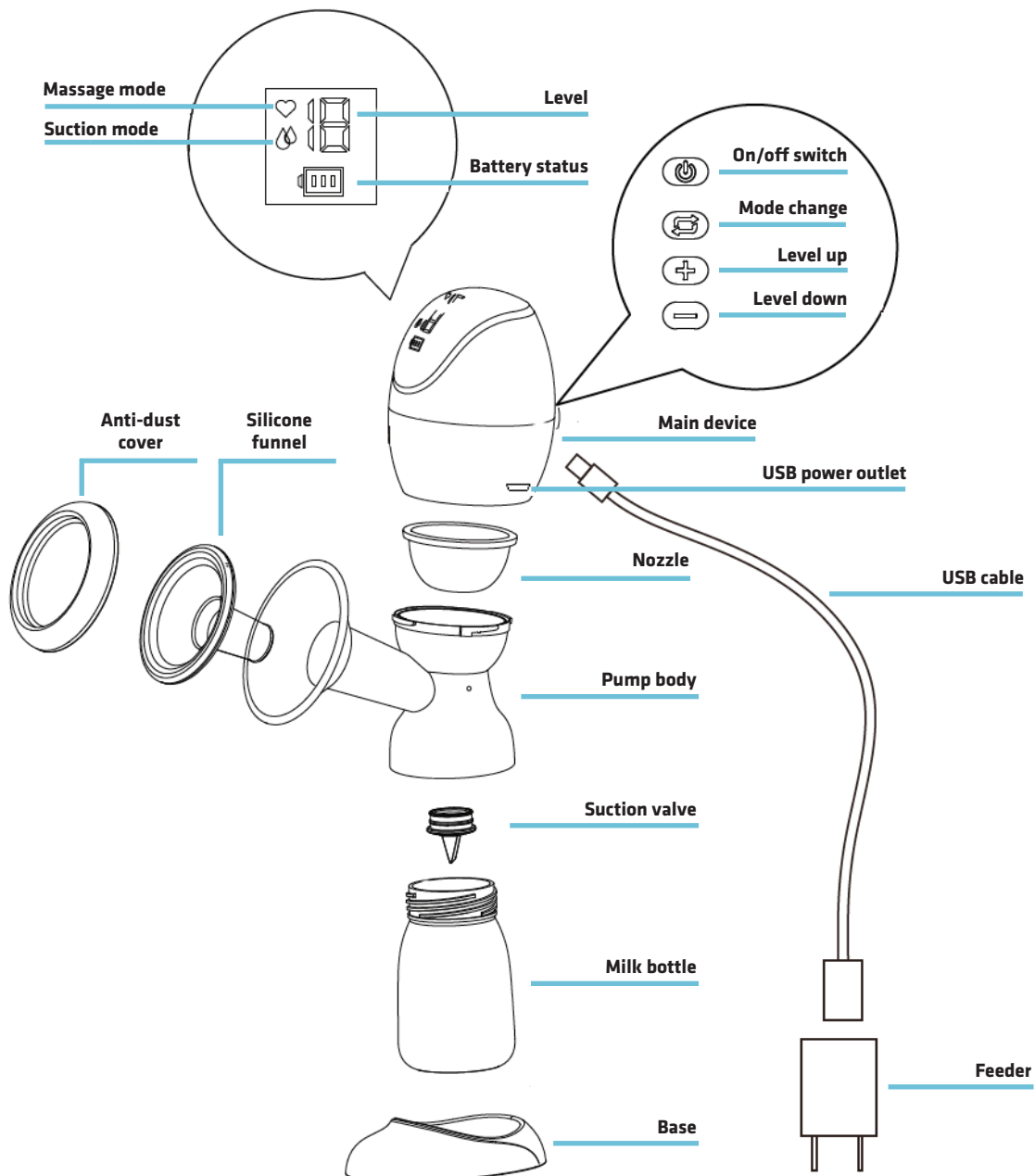
1. Do not repair or modify the device. Only an authorized service technician is allowed to perform these operations.
2. Do not switch the device on if you have noticed any damage. This applies both to a wire and plug.
3. Do not use the device if it acts up, has been dropped or immersed in water, overheats extensively, is discolored or deformed, or when other atypical symptoms occur. Contact the manufacturer's authorized service center.
4. Never leave the plugged device unattended.
5. This device is not a toy. Do not let your children play with it. Keep the device out of reach of children.
6. Use the device away from sources of heat, high temperatures, hot surfaces, direct sunlight, sources of ignition, naked flame, oils and sharp edges.
7. Do not use the device in the inflammable, explosive or toxic atmosphere.
8. Do not use chemicals to clean the device. Do not wash the main device and motor with water.
9. Switch the device off after every use.
10. To unplug the feeder, always grab a plug, never a wire.
11. Do not immerse the entire device in water or other liquids. Do not use near water.
12. Do not use the device for other purposes than designed.
13. Do not use the device and do not touch the feeder if your hands are wet.
14. Before you start cleaning or moving the device, switch it off and unplug.
15. Dry every element off carefully before the use.
16. Before and after every use, wash and disinfect all elements which fit tight to the body.
17. Use the original feeder (supplied) only.
18. If the breast pump causes discomfort or pain, switch the device off and stop using it immediately.
19. Do not use the device throughout the entire pregnancy.
20. Do not use the device if you are tired or when you sleep.

21. If you use the device but breast milk does not come out for more than 5 minutes, stop using the breast pump and contact your doctor.

BEFORE FIRST USE

1. Read the manual before you start using the breast pump.
2. Before the use, relax and make yourself comfortable on the chair.
3. Use one hand to hold a breast pump, and the other to control keys.
4. It is advisable to use the breast pump after you have a bath or shower. You can also use a hot compress or towel.
5. If you feel pain when using the device, stop using it and consult your doctor.

Device outline

BREAST PUMP MODES

1. Massage mode - 10 levels

The massage simulates child suction at the early breastfeeding stage. This mode triggers high-frequency and low-power suction.

2. Suction mode - 15 levels

The suction mode simulates child suction during breastfeeding. Suction power is higher while frequency lower.

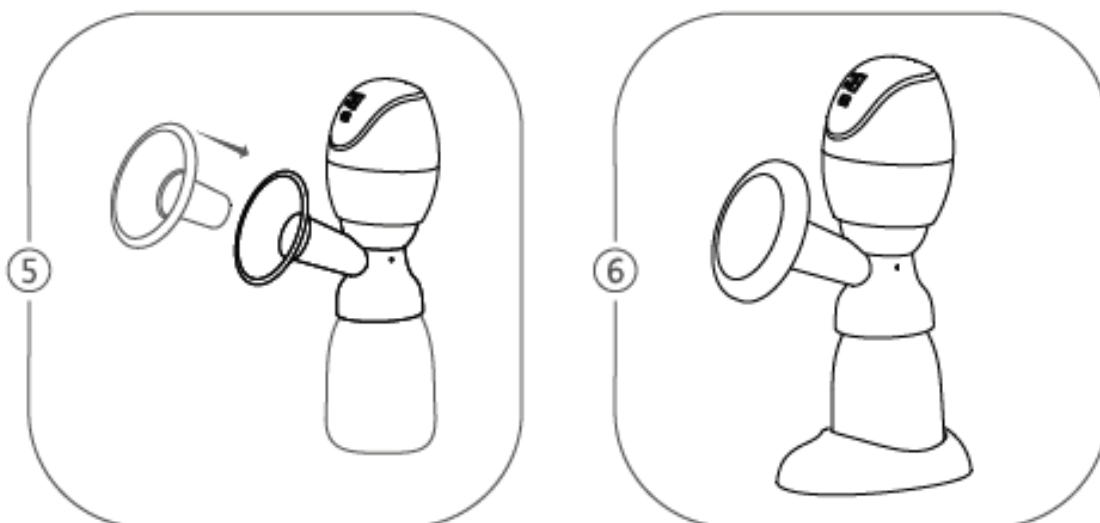
BREAST PUMP ASSEMBLY

1. Insert the suction valve into the body. Make sure the valve fits tight to avoid loss of tightness.
2. Screw the milk bottle on to the body.


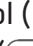
3. Put the nozzle at the top of the body; make sure it has been fixed properly and fits the edge.
4. Put the main device on the body, press and turn clockwise to secure.

5. Insert the silicone funnel into the body and put on the edge. Make sure the funnel fits the edge tight.
6. Place the breast pump on the base.

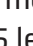HOW TO USE

1. Before you use the breast pump for the first time, disinfect the elements which contact milk (milk bottle, body, valve, nozzle and silicone funnel as specified in the manual).
2. Put the silicone funnel on the breast. Make sure a nipple is located in the middle.
3. Switch the breast pump on with (). The screen will display a massage mode symbol () and battery status symbol ().
4. Press level up () or level down () to choose a relevant level. There are 10 levels in the massage mode.
5. When milk starts coming out, press mode change key () to change suction mode ().
6. Select a relevant level. There are 15 levels in the suction mode.
7. When you finish drawing milk, press the on/off switch () to turn the breast pump off. Remove the breast pump from the breast gently.
8. Unscrew the milk bottle.
9. Clean the device as specified in the manual.

A suitable and comfortable level is selected on a case-by-case basis and does not depend on the suction power.

CHARGING

The breast pump has a built-in battery.

1. When the battery is flat, the battery status symbol will start flashing. Stop using the breast pump then.
2. Switch the breast pump off.
3. Connect the USB cable to breast pump's USB power outlet and the other end to the USB feeder.
4. When charging, the battery status symbol shows charging.

5. When the battery is full, the battery symbol is on and does not flash.
6. You can use the breast pump when charging the battery but please be highly careful.

CLEANING AND DISINFECTING

1. When you finish using, disassemble the breast pump and clean the main device with a dry and soft cloth.
2. Before the first use and after every use, disinfect all elements contacting milk (milk bottle, body, valve, nozzle and silicone funnel).
To disinfect, immerse all elements in the boiling water and leave for 10 minutes. Make sure all elements have been immersed in water and none of them touches a dry part of the vessel as they may melt.

NOTE

The main device, USB cable and feeder must not be disinfected and cleaned with water. Only accessories having a direct contact with milk must be disinfected. When cleaning, be careful because damaged elements may cause the breast pump to malfunction.

The pictures are for reference only. The real design of the products may be different from the pictures presented.

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