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Introduction

Dear customer!

Thank you for placing trust in us and choosing Lionelo.

Our products are manufactured with utmost care to provide your child with comfortable use and safety. Thanks to the use of high-quality materials and modern solution, we provide you with a product perfect for everyday use.

Should you have any remarks or questions concerning the product, please do not hesitate to contact us:

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IMPORTANT


Please read this manual before you start using the product to see all its functions and use them in accordance with intended use.

A failure to follow the safety instructions and intended use may lead to damage to the device.

Keep this manual for further reference.

1. Do not repair or modify the device. Only an authorized service technician is allowed to perform these operations.
2. Do not switch the device on if you have noticed any damage. This applies to a wire and plug.
3. Do not use the device if it starts acting up, has been dropped or immersed in water, overheats extensively, is discolored or deformed, and when other atypical circumstances occur. Contact the manufacturer's authorized service center.
4. The device is intended for home use only. Do not use it outdoors.
5. Never leave the charging device unattended.
6. This device is not a toy. Do not let children play with it. Keep out of reach of children.
7. Do not grab a wire to pull or move the device. Do not place any items on it. Do not pull the wire through sharp edges of walls and furniture, and do not wrap it too tight.
8. Use the device away from sources of heat, high temperatures, hot surfaces, direct sunlight, sources of ignition, naked flame, oils and sharp edges.
9. Do not use the device in the inflammable, explosive or toxic atmosphere.
10. Do not use the chemicals to clean the device.
11. After every use, switch the device off and unplug. To do so, grab a plug, not a wire.
12. Do not immerse the entire device in water or other liquids. Do not use near water.
13. Do not use the device for other purposes than designed.
14. Before you start cleaning or moving the device, switch it off.
15. Do not use the device and do not touch the feeder if your hands are wet.
16. Before you start cleaning or moving the device, switch it off and unplug.
17. Do not block the inlet and outlet of the device.
18. Disinfect the nebulizer before the first use. If you use the nebulizer constantly, disinfect it at least once a day.
19. Clean the nebulizer after every use.
20. The feeder supplied is not intended for charging batteries; the nebulizer does not have a battery charging function.
21. If you do not use the nebulizer, unplug it and take the battery out.

NEBULIZER OUTLINE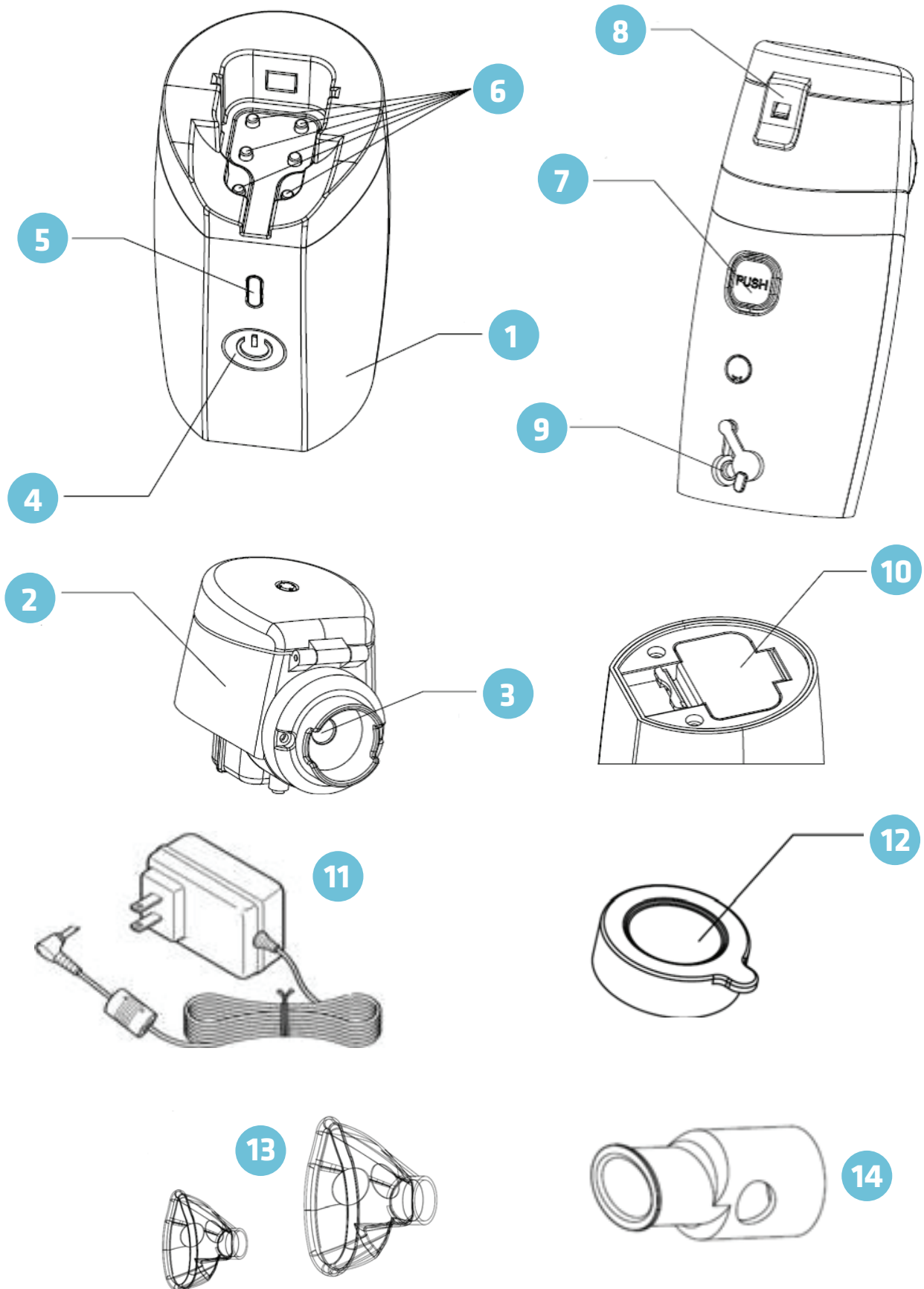


1. Main body
2. Drug chamber
3. Membrane
4. Touch switch
5. Power LED
6. Electrodes
7. Chamber eject key
8. Drug chamber fastener
9. Power supply and flap
10. Battery holder (2xAA)
11. Feeder
12. Membrane plug
13. Masks
14. Mouthpiece

HOW TO USE NEBULIZER

Batteries

1. Open battery holder.
2. Insert two AA batteries in the holder. Position them in accordance with (+ and -) polarization marked on the cover.
3. Close a battery holder cover.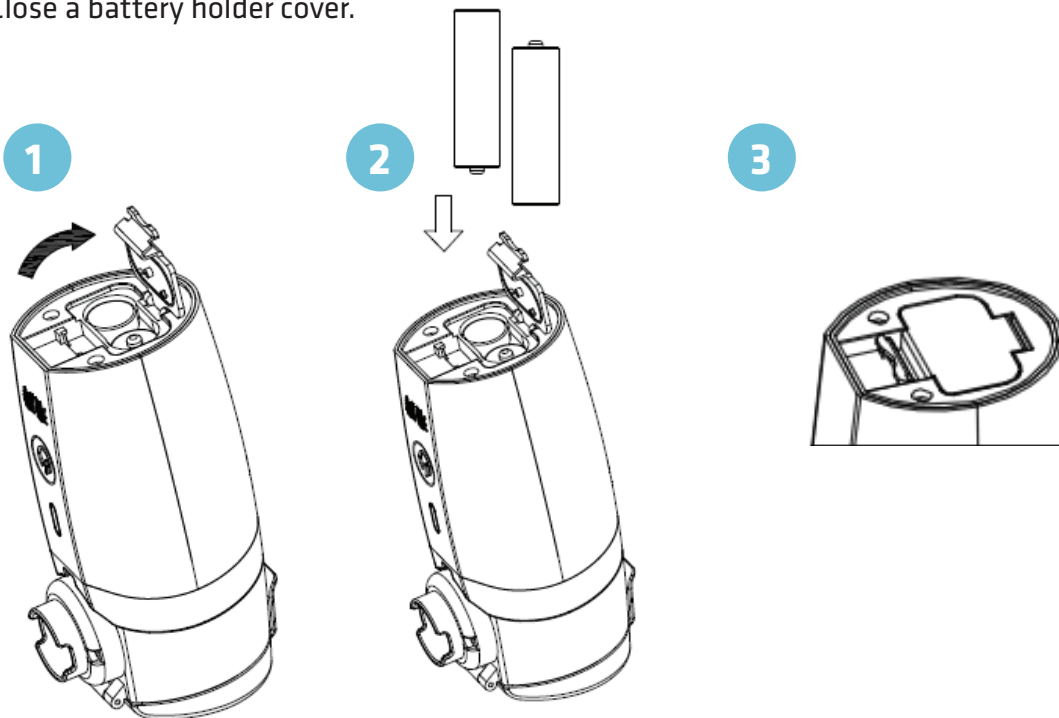

The nebulizer may use alkaline batteries, nickel-metal-hybride (NiMH) batteries and storage batteries. Depending on the type of batteries and their condition, the nebulizer may operate continually for up to 90 minutes.


NOTE

1. When the battery voltage is low, a red light starts flashing. Exchange batteries.
2. The feeder supplied and nebulizer do not have battery charging option.
3. If you do not use the nebulizer, unplug it and take the batteries out.
4. Never mix batteries from various manufacturers, various types and new with old ones.
5. Worn-out batteries are considered hazardous waste and must be disposed of in relevant containers or handed to battery collection center.

Feeder

NOTE: Do not use the device if the batteries are inside and feeder is connected at the same time.

1. Take the flap out and insert the feeder plug in the socket.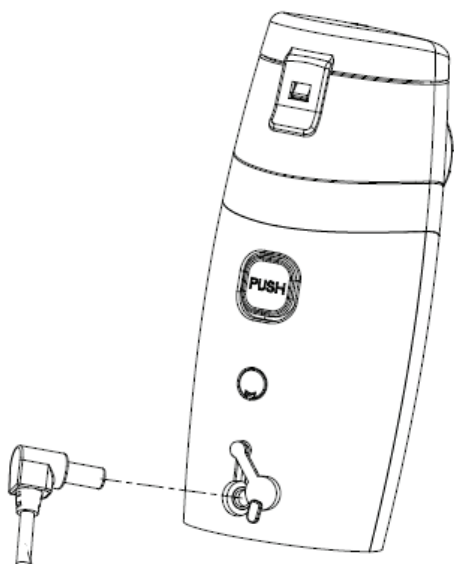


2. Connect the feeder to the electric socket.
3. Before you disconnect the feeder from the nebulizer, unplug the feeder from the socket in the first place.


How to assemble the main body and fill drug chamber in

Before the first use or if the nebulizer has not been used for a long time, disinfect the drug chamber, membrane plug, masks and mouthpiece. The disinfection guidelines can be found at the end of this manual.

1. To separate the drug chamber from the main body, hold the button in the back of the casing and pull the chamber upwards.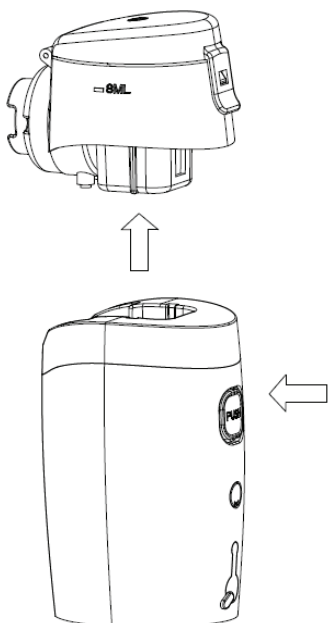


2. Open drug chamber.
3. Pour the drug into the chamber. Maximum volume up to 8ml.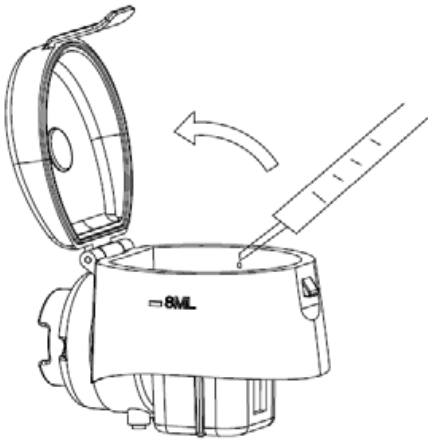

4. Close the chamber. If you can hear a distinctive click, the chamber has been closed properly.


NOTE: Make sure the chamber has been closed properly – this guarantees watertightness.

5. Place the chamber in the main body and press gently. You can hear a distinctive click if the chamber has been placed properly.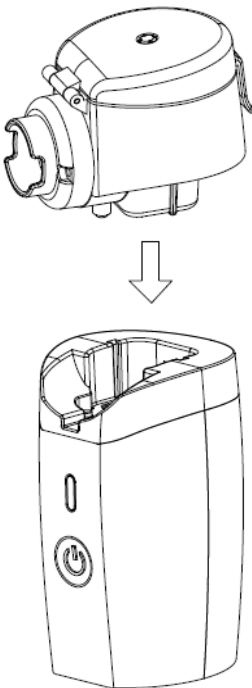

NOTE: Keep the main body clean. Dry the body as well as the chamber surface before the assembly and use.

6. Remove the membrane plug, next put on the mask or mouthpiece.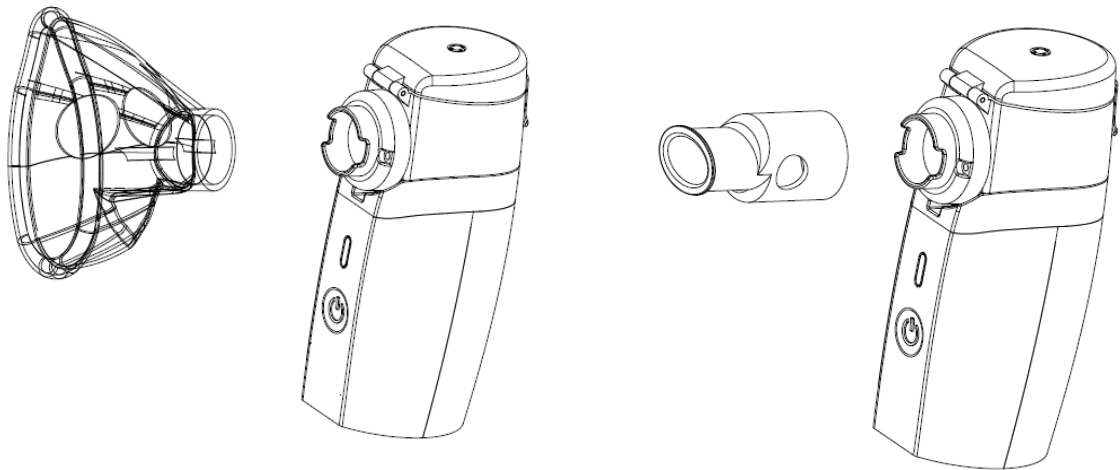

7. Gently touch the switch to start nebulization.


Nebulization

NOTE: Use the type and amount of drug in accordance with doctor's prescription.

1. Switch the nebulizer on by touching the switch with a finger. Green LED will go on.

NOTE: If red LED flashes after you have switched it on, it means that the drug chamber is empty, chamber has been assembled improperly or batteries are flat.

2. Check if the aerosol goes out of the membrane after starting – if this is the case, the device operates properly.
3. Gently tilt the nebulizer, as presented below. In this position a vibrating filter is immersed in the drug and nebulization can be initiated. After you have taken this action, you can use the nebulizer at any angle. If nebulization stops and the drug is in the chamber, tilt the nebulizer again to immerse the filter in the drug.

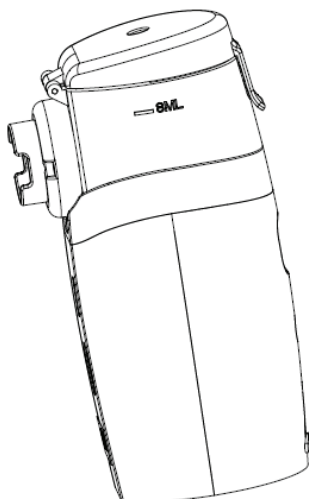4. Put the mouthpiece in your mouth or the mask on the lips and nose.

Be relaxed, breathe slowly and deeply. Every time you inhale, hold the breath and exhale by taking the mouthpiece out or removing the mask.

Do not exhale air directly into the mouthpiece or mask.

Do not breathe too fast, take a short break between breaths.

NOTE: If you use high-viscosity drugs, nebulization degree may be reduced. When a large amount of drug accumulates on the membrane, nebulization may be suspended. Gently clean the membrane with a lint-free cloth. **To clean the membrane, do not use cotton swabs or pins because they may damage the membrane.**

5. When the nebulizer has used the drug up, red LED starts flashing. After inhalation has finished, switch the nebulizer off and unplug (if connected).

CLEANING, STORING AND MAINTENANCE

Cleaning after every use

NOTE: If you do not clean and disinfect the device regularly and properly, microorganisms may accumulate on the device and infect it.

Do not touch the membrane with finger, swabs and sharp items.

After every use, clean the drug chamber, membrane plug, mouthpiece and masks. We suggest that you pour a distilled water into the drug chamber after inhalation and start nebulization to remove the drug from the chamber completely.

1. Disassemble the nebulizer.
2. Rinse the drug chamber, membrane plug, mouthpiece and masks under running hot water.
3. Shake surplus of water off and dry parts in the fresh air on a dry and clean towel.
4. Make sure all parts are fully dry and clean before you hide them or use again.

Cleaning at the end of the day

1. Wash the drug chamber, plug, mouthpiece and mask in a warm water, use a soft dish-washing liquid.
2. Rinse all parts under running hot water carefully.
3. Shake the surplus of water off and dry parts in the fresh air on a dry and clean towel.
4. Make sure all parts are fully dry and clean before you hide them or use again.
5. Wipe the main body and electrodes with a wet cloth without detergent, next wipe them dry.
6. Assemble the entire nebulizer.
7. Store the nebulizer in a clean and dry place.

Daily disinfection

Drug chamber, membrane plug, mouthpiece and masks must be disinfected:

1. Before first use.
2. When nebulizer has not been used for a long time.
3. At the end of each day.

Before you start disinfection, clean the elements as specified above.

NOTE: To disinfect, do not use chemicals as they may damage the elements and above all metal membrane.

How to disinfect with an acetate solution:

1. Make a solution with three parts of hot water and one part of white distilled vinegar (synthetic).
2. Immerse elements in the solution for 30 minutes.
3. Rinse all elements under running hot water.
4. Shake surplus of water off and dry elements in the fresh air on the dry and clean towel.
5. Make sure all parts are fully dry and clean before you hide them or use again.

How to disinfect through boiling:

1. Boil distilled water.

NOTE: If you boil (disinfect) elements in the non-distilled water (tap water, bottle water), elements' life may get shorter.

2. Insert all elements in the boiling water. Boil for up to 20 minutes.
3. Take all elements out, shake the surplus of water off and dry elements in the fresh air on the dry and clean towel.
4. Make sure all parts are fully dry and clean before you hide them or use again.

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